
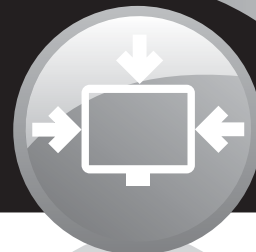
 **ALCAD**

CHC-240

9400012

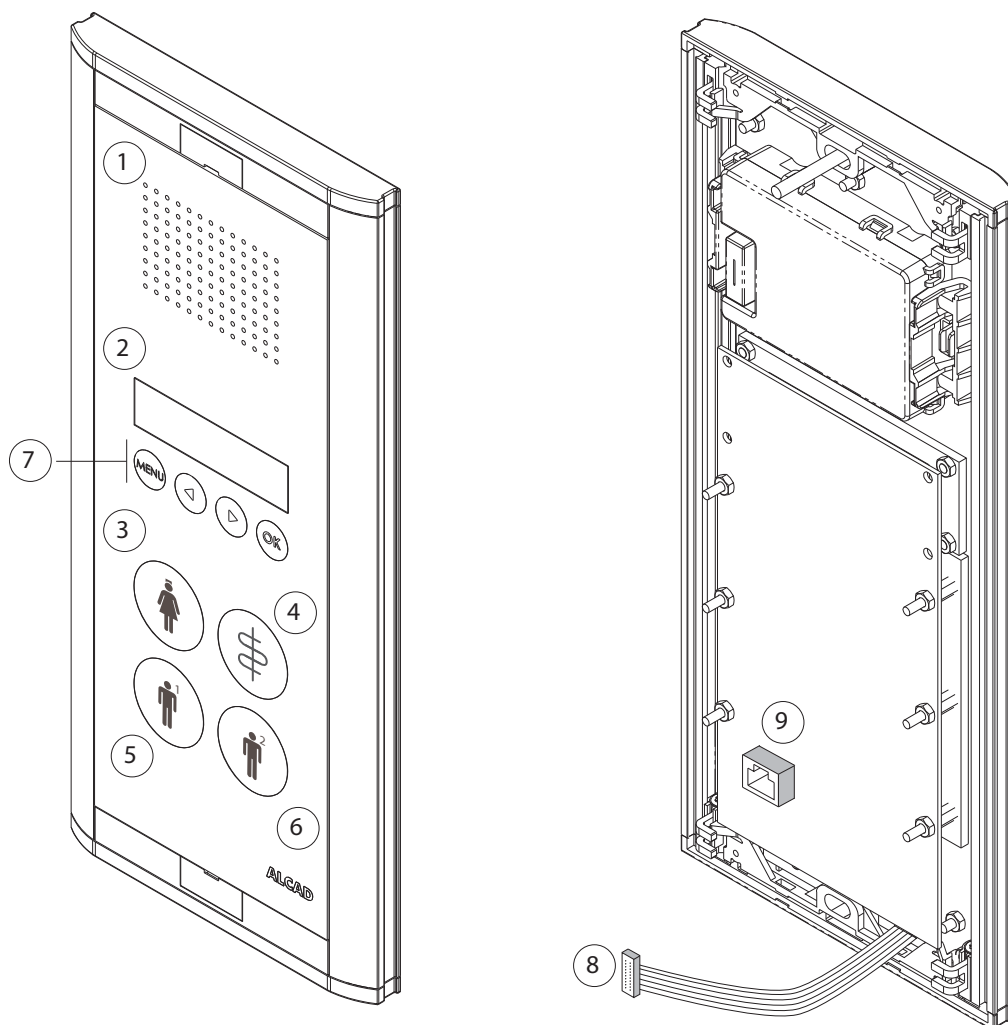CENTRAL DE HABITACIÓN VERTICAL
VERTICAL ROOM POST
POSTE DE CHAMBRE VERTICAL

*Dimensiones (mm): 127 (An) x 290 (Al) x 15 (Pr)

*Dimensions (mm): 127 (Wi) x 290 (He) x 15 (De)

*Dimensions (mm): 127 (La) x 290 (Ha) x 15 (Pr)

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To mount the flush-mounted box and connection board, follow the ACC-141 data sheet.

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Make the electrical conexions for the room elements from the 941serie to the connection board following the instructions of the manager software.

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Connect the connections board to the CHC-240 through the 20-way connector supplied. Use ethernet cable category 5e or higher to connect the CHC-240 to the related switch.

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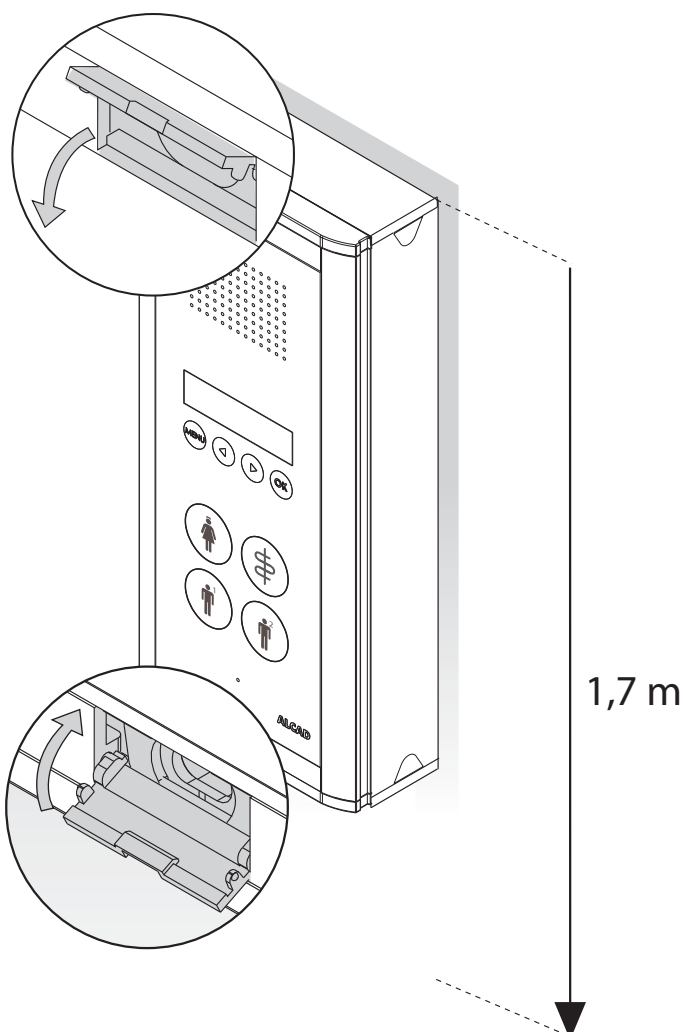
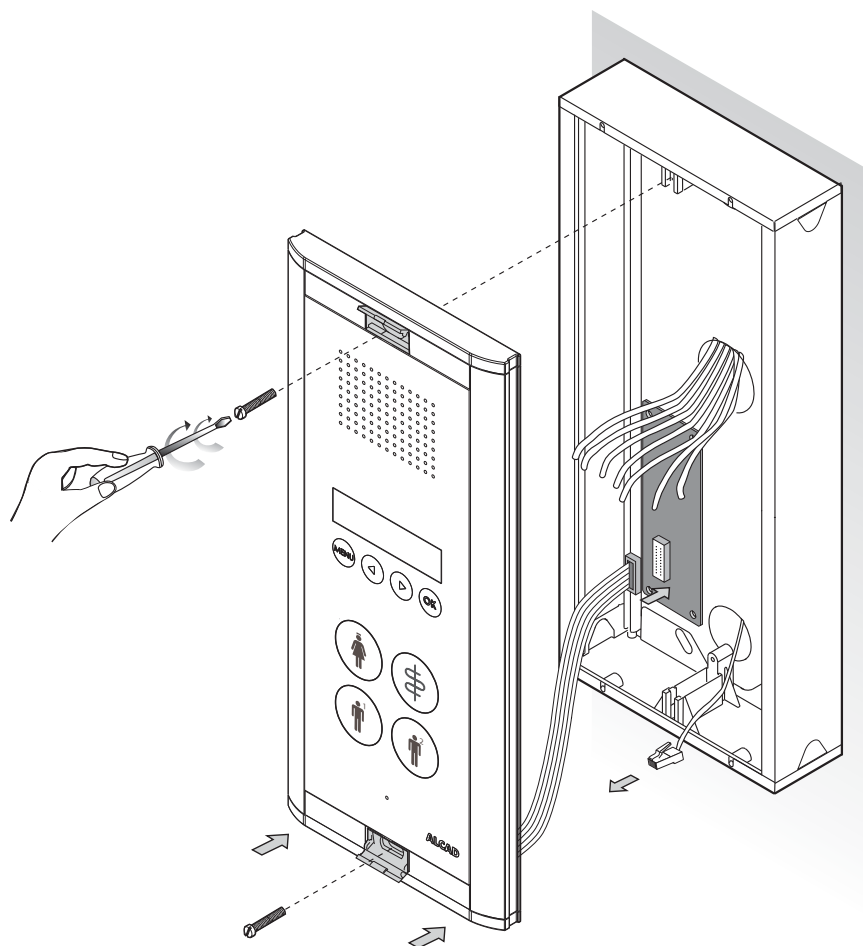
Fix the CHC-240 to the flush-mounted box using the supplied screws.

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Mount the surface wall-mounted box and connections board according to the directions in the ACC-140 data sheet.

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Make the electrical conexions for the room elements from the 941serie to the connection board following Manager directions.

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Connect the connections board to the CHC-240 through the 20-way connector supplied. In turn, connect via ethernet cable cat5e or higher. The CHC-240 to the related switch.

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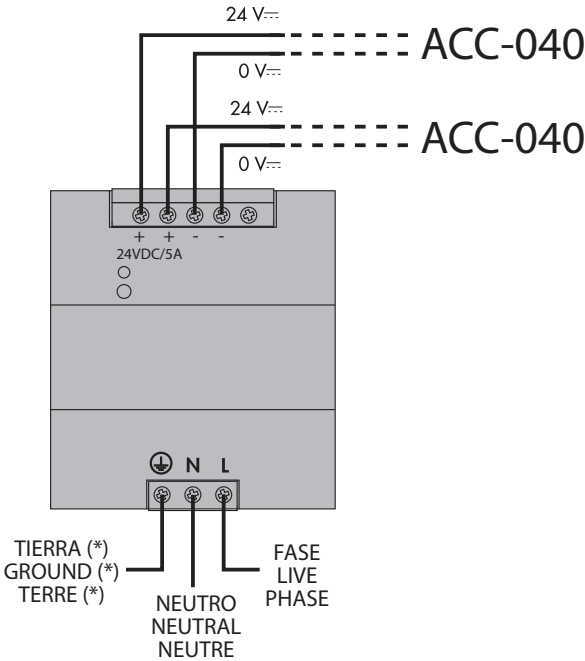
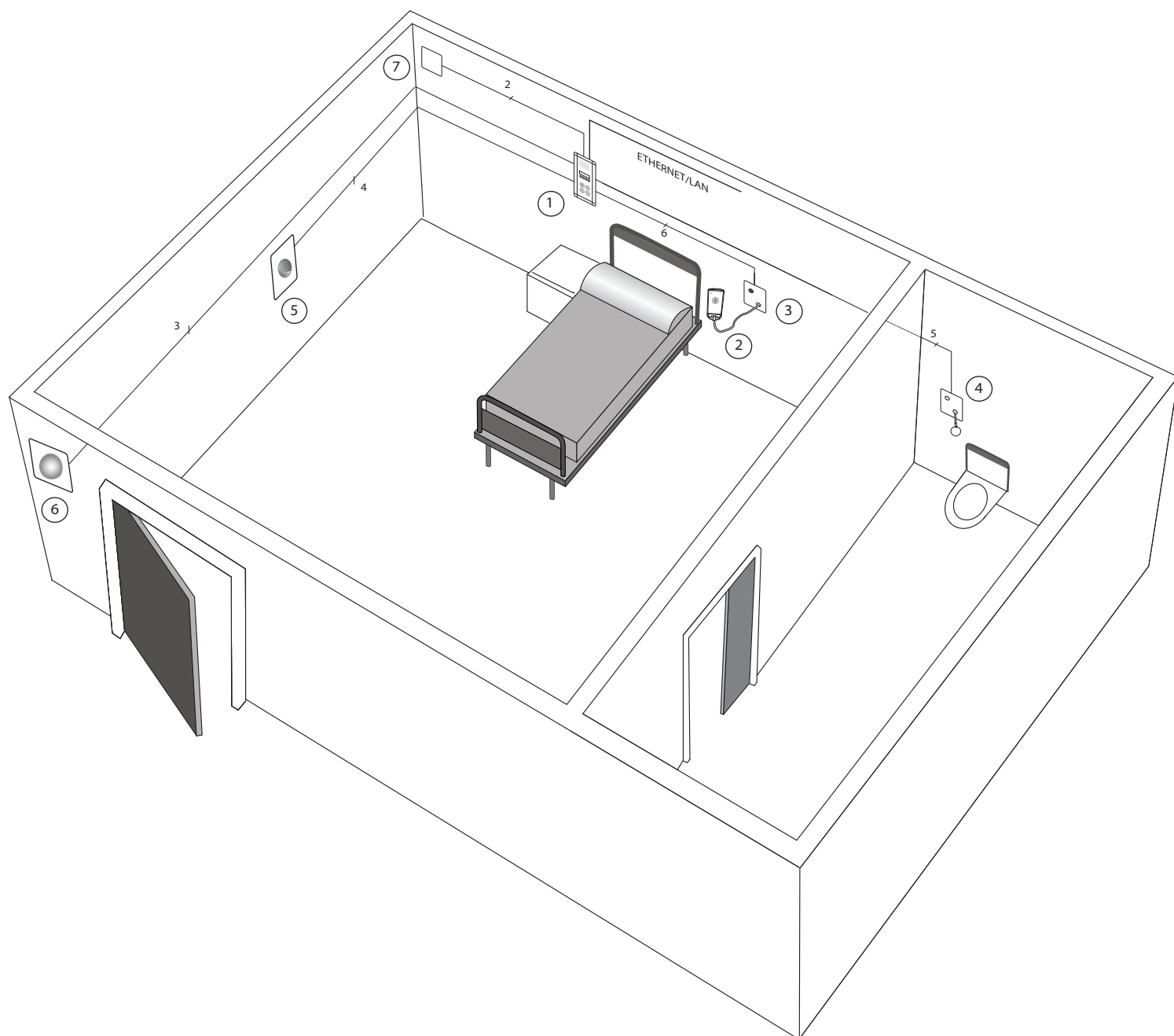
*Recommended power cable section: 1,5 mm²

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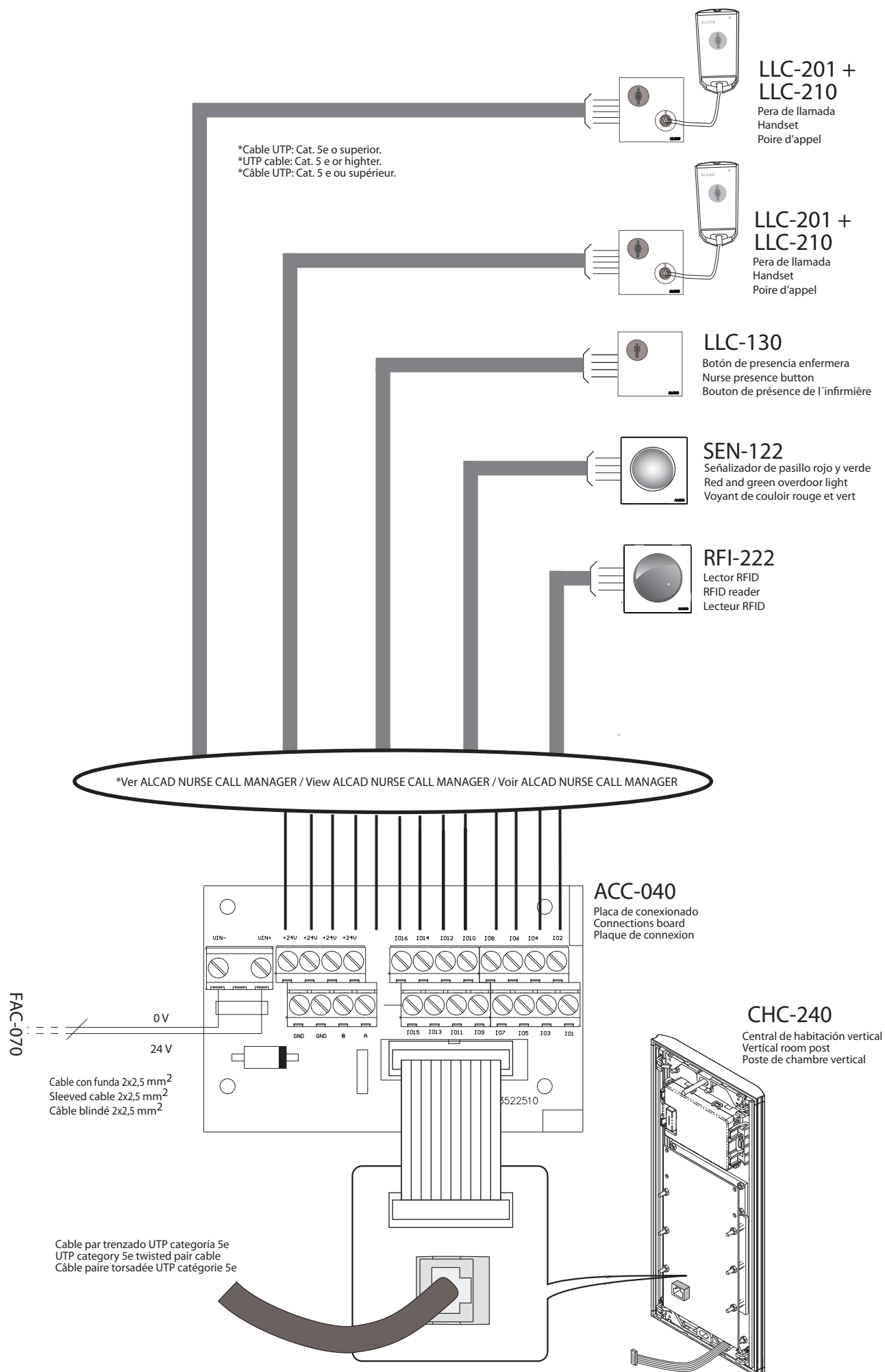
*Recommended cable section for connecting peripherals: 0,25 mm² or UTP cable

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Safety Instructions

Do not place the equipment where water can drip or splash onto it. Do not place objects containing liquid, such as glasses, on the equipment. Do not place sources of naked flame, such as burning candles, on the equipment. Do not block the ventilation slots of the equipment with objects such as newspapers, curtains, etc. When installing the equipment, leave some free space around it to provide adequate ventilation. Install the equipment in such a way that the mains supply plug or the connector of the equipment can be easily reached.

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Product Name: VERTICAL ROOM POST

Product Number(s): CHC-240

Product Description: Short Range device

Product Option(s): INCLUDING ALL OPTIONS

is in conformity with:

EMC: EN 61000-6-3:2007 Safety: EN 60065:2002
EN 61000-6-1:2007

Hereby, Alcad, S.L. declares that the equipment CHC-240 is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

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Supplementary Information: To comply with these directives, do not use the products without covers and operate the system as specified.

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